DUKWOO
(Safe, Sharp, Soft)

Embryo Transfer Catheter

(3S-E.T.Cath-3.5)

3.5Fr-A

3.5Fr

(주)덕우메디칼
DUKWOO MEDICAL CO.,LTD.

3S-E.T.Cath (3S-Embryo Transfer Catheter)

(주입-배액용튜브 · 카테타 : Introduction-drainage tube and catheter)

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■ purpose

- injection sperm or in-vitro fertilized embryo into the womb
- discharge the body fluids from the coelom and use for clinic test
- inject anti-adhesion to prevent adhesion for operated region
- inject the medicinal substance into the womb, etc

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■ special feature

- the surface of the tube is very smooth and the tip is manufactured elaborately to prevent injury while injecting to the womb
- you can adjust the length according to need
- the tube is kink resistance while inject, or you may bend with your intended angle
- marking for using the adequate length when inserting

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■ caution to use

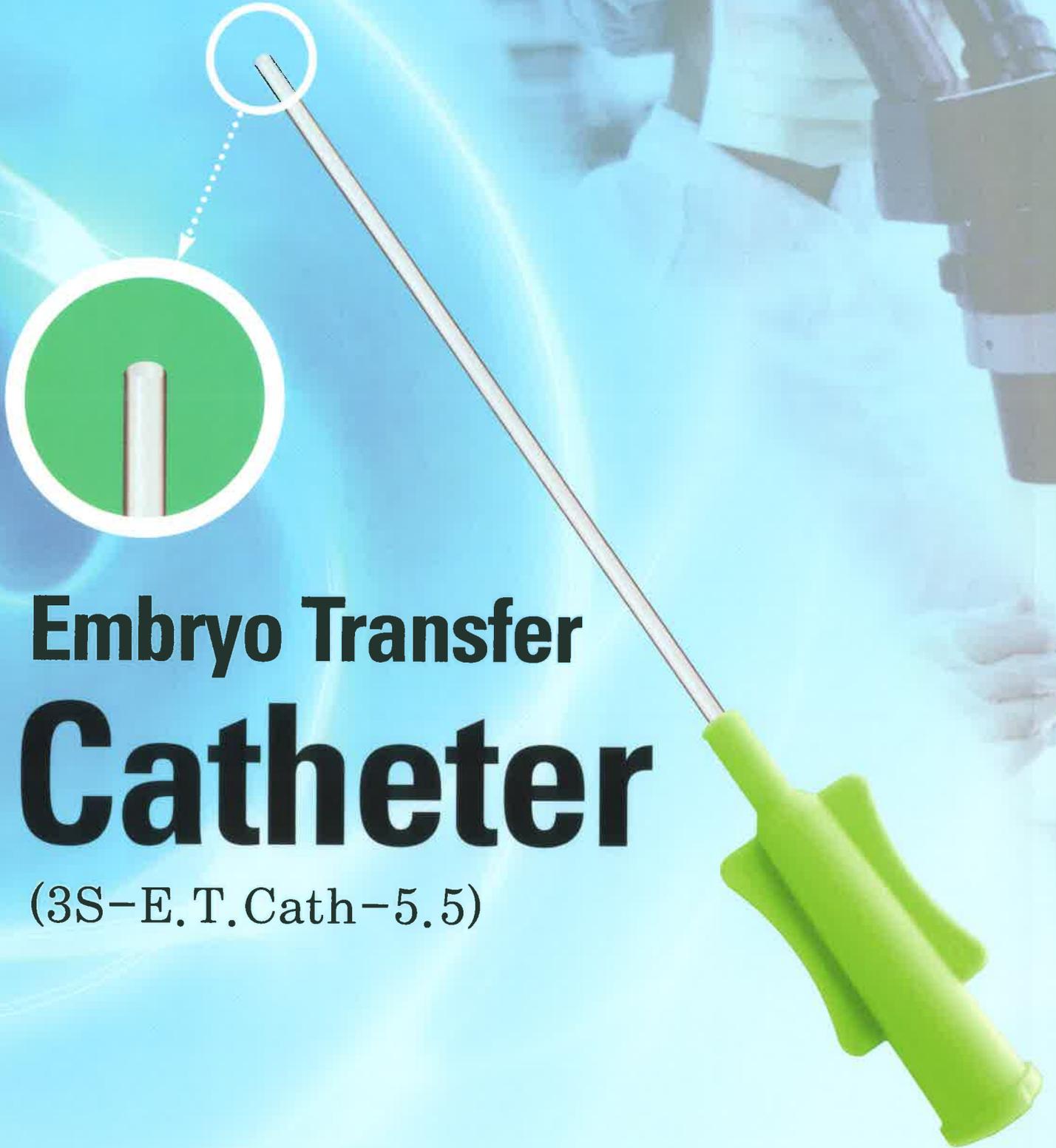
- not for reuseable
 - check the package of the goods(3 years from manufacturing date)
 - check the package of the goods
 - after opening the package, use it immediately
 - do not insert the mark to human body when the marked product is used
 - if the patient has a cervical infection ,never use
- *Be prepare the disposable syringe with 1ml or 2.5ml(sterized one)

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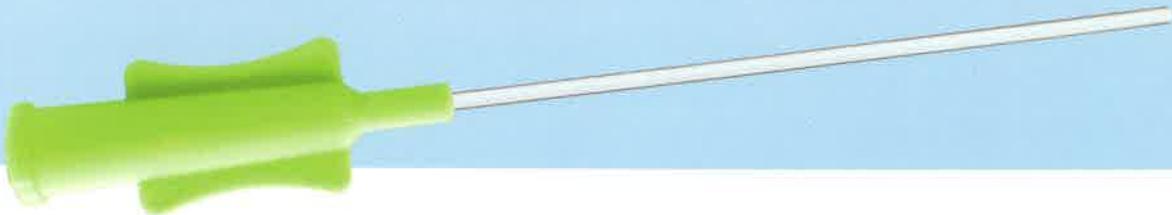
■ purpose

- inject the anti-adhesion to prevent adhesion for operated region
- injecting sperm or in-vitro fertilized embryo into the womb
- discharge the body fluids from the coelom and use for clinic test
- inject the medicinal substance into the womb, etc

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■ special feature

- the surface of the tube is very smooth and the tip is manufactured elaborately to prevent injury while injecting to the womb
- inject the medicinal substance to the wounded part in coelom, Especially it is possible to injecting the anti-edhesion directly
- the tube is kink resistance while inject, or you may bend with your intended angle

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■ caution to use

- not for reuseable
- check the expiration date of the goods(3 years from manufacturing date)
- check the package of the goods
- after opening the package, use it immediately
- if the patient has a cervical infection, never use
- *Be prepare the disposable syringe with 1ml or 2.5ml(sterized one)