

# Ovum Pick-up Needle (Biopsy Kit)

## Single Lumen Ovum Pick-up Needle

### Application

- This single lumen needle is used for picking up ovum and in vitro fertilization (I.V.F.)
- Available in 17, 18, 19Gauge. The color coded to indicate cannula specification.
- Needle are available in regular wall.


### Caution

- Single use only.
- Sterile : Unless individual package has been opened or damaged.

### Storage

- Keep away from the place of high temperature and humidity.  
Please keep custody at normal temperature.

### Description

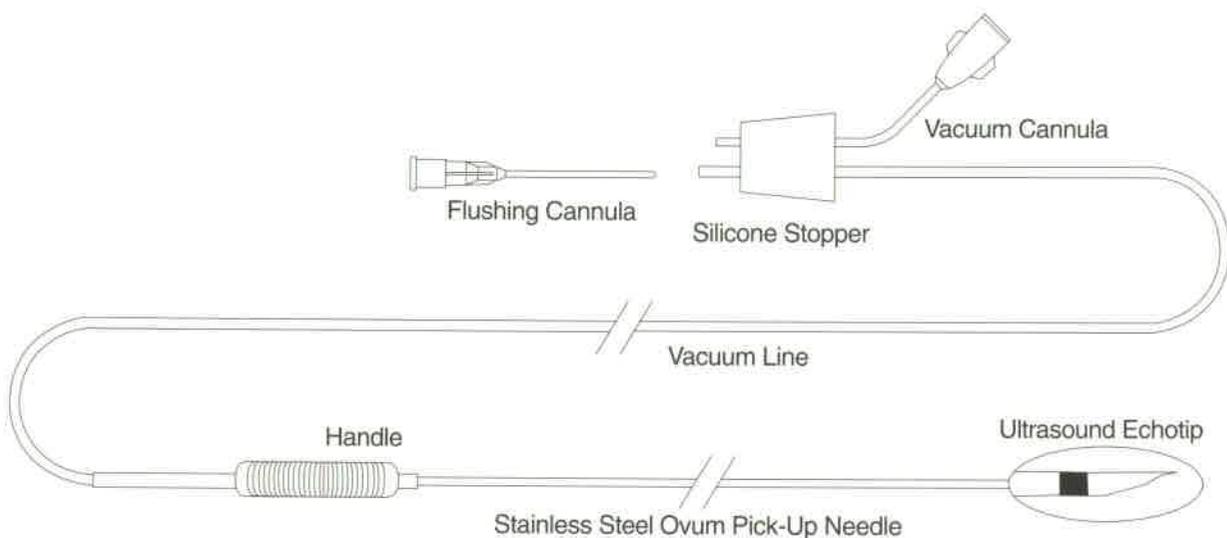

- Standard front bevel tip provides the needle with the ability to harvest from smaller follicles.
- Standard non-coring bevel tip prevents tissue entering lumen.
- Translucent FEP vacuum line is 60cm long from needle to silicone stopper.
- Flushing the needle and vacuum line by attaching flushing cannula to syringe and inserting cannula into the flared end of the vacuum line to silicon stopper.
- Spically treated tip enhances ultrasound echo tip of needle.
- Sterilized by E.O. Gas.



## Order Information

Specification	Needle Size(mm)		FEP Vacuum Line (Length : cm)	Color	Package
	O.D.	I.D.			
16G	1.65	1.36	60	Blue	Individual Pack : One side-medec paper, other side-p.p.film 10pcs/carton box
17G	1.47	1.21	60	Purple	
18G	1.27	1.00	60	Pink	
19G	1.07	0.83	60	Brown	
20G	0.91	0.67	60	Yellow	

※ We are regularly producing 17G and 18G.

# Ovum Pick-up Needle (Biopsy Kit)

## Ovum pick-up system

### Preparation before use

- Please confirm the package of ovum pick-up needle set.
- Confirm the manufacturing date, validity and whether the packing state is thoroughly sealed or not. If the packing and product have been damaged, please do not use it and use the perfect one.

### Operating method

- Connect the hub of vacuum cannula to the connector of vacuum pump & regulator firmly.
- Take handle of the needle and insert the needle tip carefully to follicle through the guide inserted previously into womb confirming etching mark of the needle end.
- When the correct position of needle tip is confirmed, vacuumize the falcon tube which is used for collecting ovum, translucent FEP tubing and inside of needle then collect the ovum.
- After collecting ovum, take out the ovum pick-up needle from follicle.
- Flush the inside of translucent FEP tubing and needle with flushing cannula and pyrogen free water if necessary.

